


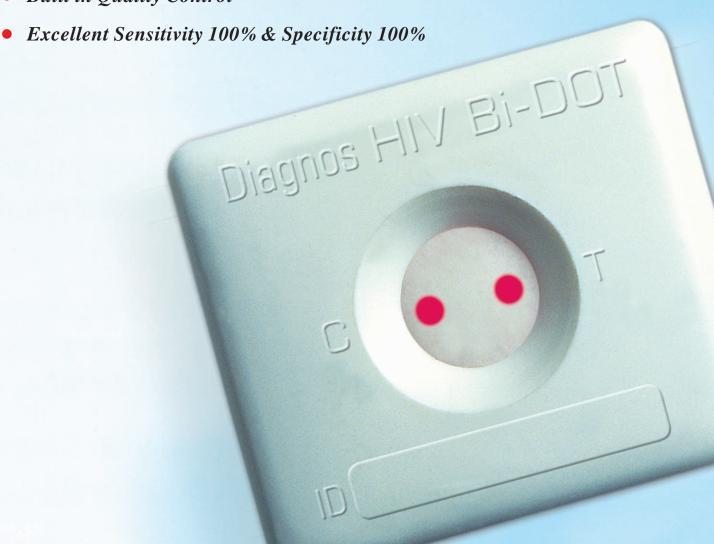

Rapid Visual Test for the Detection of HIV-1 (including Group 'O' & subtype 'C') & HIV-2 in Human Serum or Plasma


- Results within 3 minutes
- Flowthrough Technology
- No Instrument required
- Built in Quality Control



In flow through technology antibodies to HIV-1/2 bind to the immobilized HIV antigen. Sample also contains other non-specific antibodies.


The non-specific antibodies are washed away by washing.



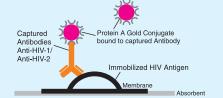

Protein A conjugate binds to the Fc protion of Anti-HIV-1/Anti-HIV-2 antibodies.

After adding the Protein A gold conjugate unbound conjugate is washed away during the washing step.

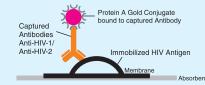








TEST PROCEDURE

- Add 3 drops of Buffer Solution.
- Add 1 drop of Patient's serum/ plasma.
- Add 5 drops of buffer solution.
- Add 2 drops of conjugate solution.
- Add 5 drops of buffer solution.
- Read Results.

Results Interpretation



REACTIVE: for HIV-1 and/or HIV-2

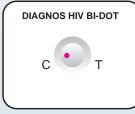

NON-REACTIVE





INVALID

Kit Presentation

10 Test Pack & 50 Test Pack

For further information, please contact:

J. Mitra & Co. Pvt. Ltd.

A 180-181, Okhla Industrial Area, Phase-1, New Delhi-110 020 - INDIA Ph: 91-11-47130300, 47130500, 26818971-73 Fax: 91-11-26818970, 26810945

R-01